
Basic Patient Care Manikin

Instructions for Use, Care and Maintenance



Model No. 150-1370 105lb.

Model No. 150-1372 35lb.

 **Simulaids**

INSTRUCTIONS FOR USE, CARE AND MAINTENANCE

BASIC PATIENT CARE MANIKIN

This manual will familiarize the user with the function, care, and maintenance of the SIMULAIDS' Patient Care Manikin. All of our manikins are designed and manufactured to provide years of effective and trouble-free use when the simple steps of care and maintenance are regularly performed.

FEATURES

Your manikin is designed to assist in the training and practice of basic patient care, handling, and movement. It features fully articulated limbs and torso so that proper positioning in a bed, wheelchair, or gurney can be taught and practiced.

Patient-sensitive bathing techniques and general hygiene procedures can be performed (do not submerge in water).

Bandaging and dressing may be performed on head, torso, limbs, hands, and feet.

The wig can be shampooed, styled, and combed.

CARE AND MAINTENANCE

Your Patient Care Manikin may be washed in hot soapy water. The use of solvents or strong cleansers is not recommended. However, an "all purpose" cleanser (i.e. 409 or FANTASTIC) may be used. The genitalia may be washed in the same manner.

NOTE: To prevent deformation of body parts take care to:


1. Store only in supine (*face up*) position.
2. Make certain nothing is stored on top of manikin.
3. Store in area with temperature below 95° F.

CONSTRUCTION

Body segments are suspended on a skeleton of steel rod, bar stock and cable. The metal is zinc plated to resist corrosion and should provide years of use. Some enlargement of the body joints may develop with extended use, without affecting the serviceability of the manikin.

Your Patient Care Manikin may be used in very low temperatures, but if used in this way you can expect the cast vinyl cover to become stiff. The cover will return to normal flexibility once it is returned to a higher temperature.

LEG ASSEMBLY DIAGRAM

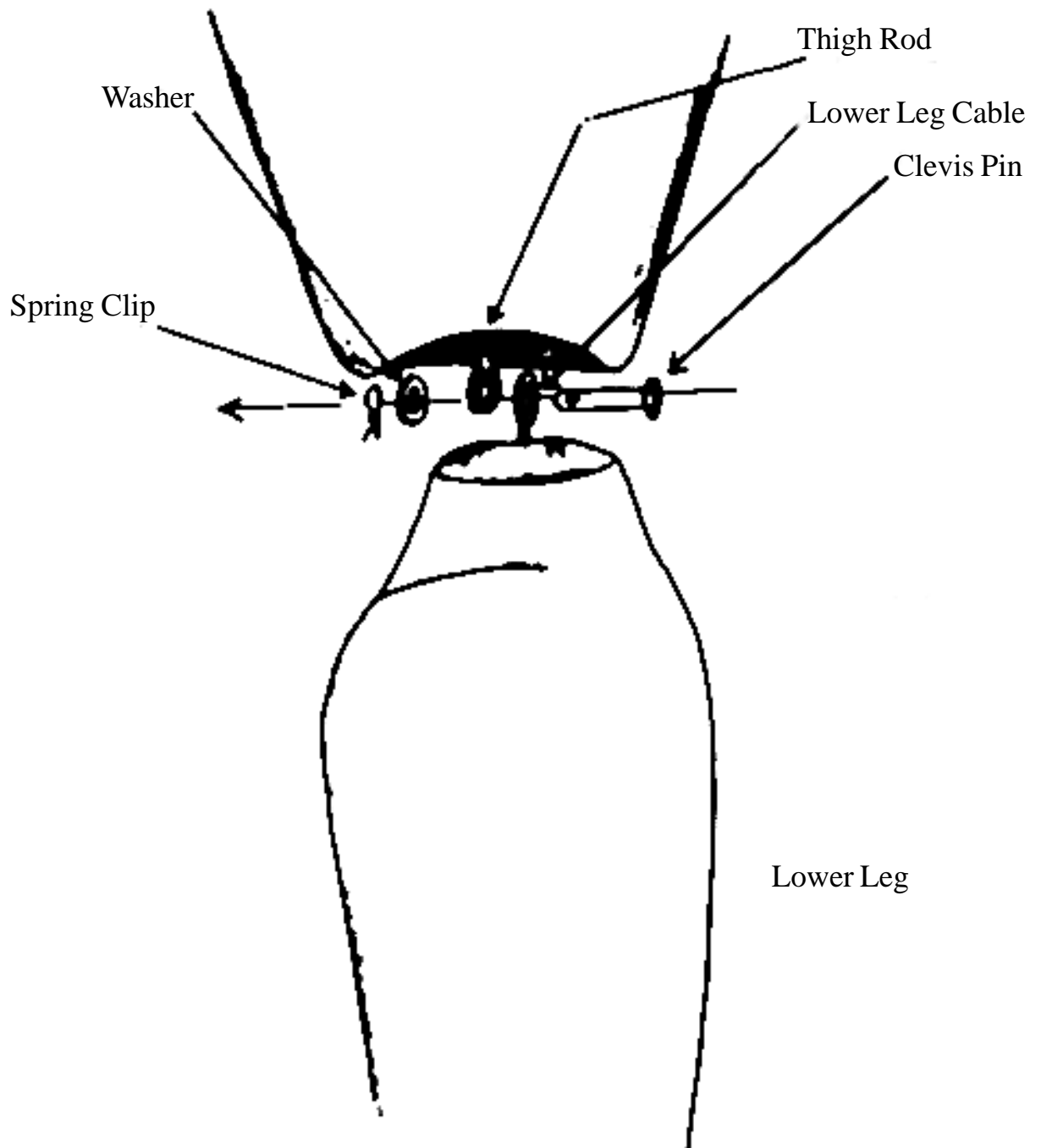

WARRANTY Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.


ACCESSORIES

Wig - No. 0537

Male Genitalia - No. 1376

Female Genitalia- No. 1378

Carry Bag - No. 1373

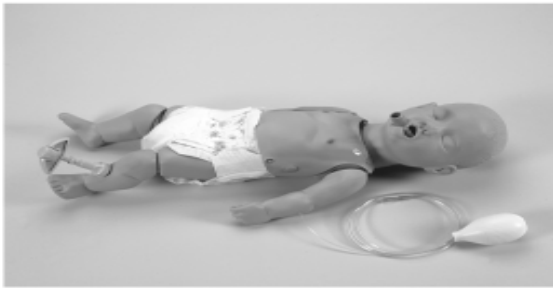

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com