


Assembly Instructions for PROSCOPE™ Teaching Stethoscopes (Models #671 and #661)

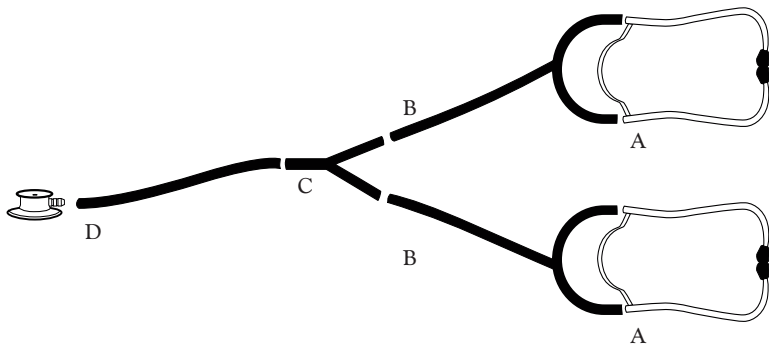

TO ASSEMBLE

- A. Attach “Y” tubing to each set of the binaurals (660-05) as shown in the illustration.
- B. Attach free end of each “Y” tubing to the double pair end of the “Y” connector (#611-20).
- C. Connect one end of the straight tubing (#671-05TBK) to single end of the “Y” connector.
- D. Connect remaining end of the straight tubing to stem of chestpiece (#660-01 or #670-01).

Make sure all tubing connections are firm.

PROSCOPE™ Teaching Stethoscopes

(Models #671 and #661)


Replacement Parts Included

Part Number	Description	Qty.
660-01	PROSCOPE™ Nurse Chestpiece	1*
670-01	PROSCOPE™ Dual Head Chestpiece	1*
611-20	“Y” Connector	1
671-05TBK	PVC Tubing, Straight, 10”, Black	1
660-05T	PVC “Y” tubing, 22”	2
660-05	Binaurals	2

*NOTE: PROSCOPE™ Model #671 includes the 670-01 chestpiece only
PROSCOPE™ Model #661 includes the 660-01 chestpiece only



 ADC
55 Commerce Drive
Hauppauge, NY 11788
U.S.A.

 SC Cattus SRL
Aleea Carpati nr.53 Sc. B Ap.64
Târgu-Mures, România, EU

tel: 631-273-9600
1-800-232-2670
fax: 631-273-9659
www.adctoday.com
email: info@adctoday.com

For Australian Consumers

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.



Medical device



Not made with natural rubber latex



Phthalate free



Consult instructions for use