



Practi-Injecta Pad™ & Practi-Anatomical Templates™ Care Instructions

The Practi-Injecta Pad allows for a safe injection practice with either air or liquid injections. The six interchangeable Practi-Anatomical Templates represent the most commonly used landmarks and injection sites. Each Practi-Anatomical Template overlays the surface of the Practi-Injecta Pad for quick visualization, realistic anatomical placement, and more authentic injection practice.

Practi-Injecta Pad Care Instructions:

1. DO NOT sterilize the injection area with any colored disinfectant as permanent staining will result.
2. Normal surface soil can be removed with mild soap and lukewarm water.
3. When injecting liquid, make sure to properly drain the pad overnight.
4. Always replace the Practi-Injecta Pad back in its storage box after use.
5. Store at room temperature, avoiding extreme heat or cold.

Draining Instructions:

Water and normal saline may be injected into the Practi-Injecta Pad without damage. Prepare the pad by puncturing several holes with a large-gauge needle. Turn over the pad and place on a towel overnight to drain. Dry the Practi-Injecta Pad and store in its original storage box.

Practi-Anatomical Template Care Instructions:

For long-lasting use following practice, the templates should be dried and stored in the protective sleeve.