


XR-Clinic Pico 4 setup manual





Set up the headset

- Charge the headset using USB cable
- Hold the power button on the right side to turn on VR headset
 - Create the boundary or chose the stationary one
- Connect the headset to WI-FI and install system updates

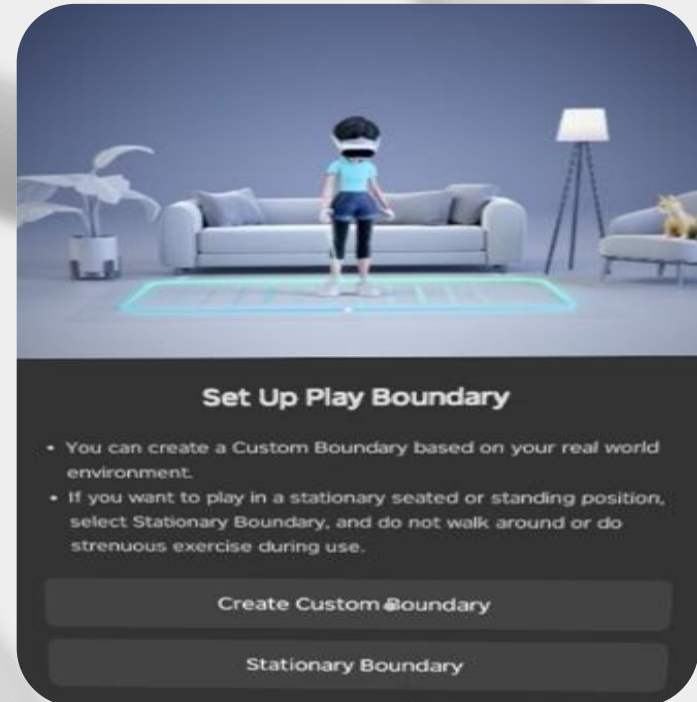



Install the application

- Go to the Library - Not Installed
- Click on the “Beta” icon to download XR-Clinic
 - Go to the Library - Apps
- Click on the XR-Clinic application to launch it

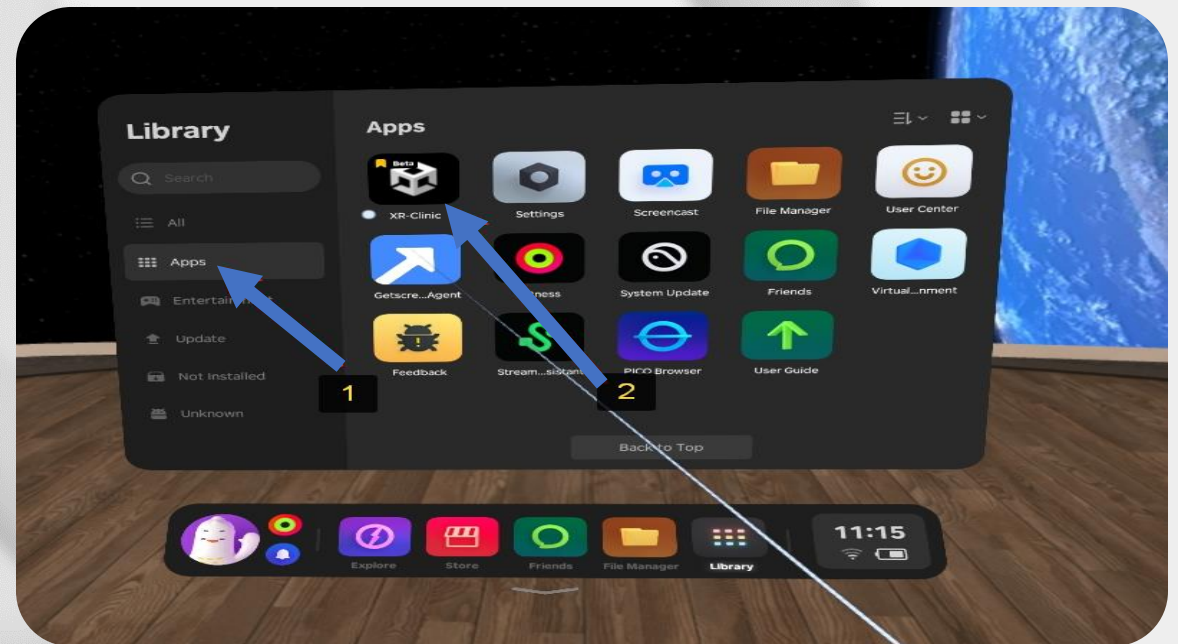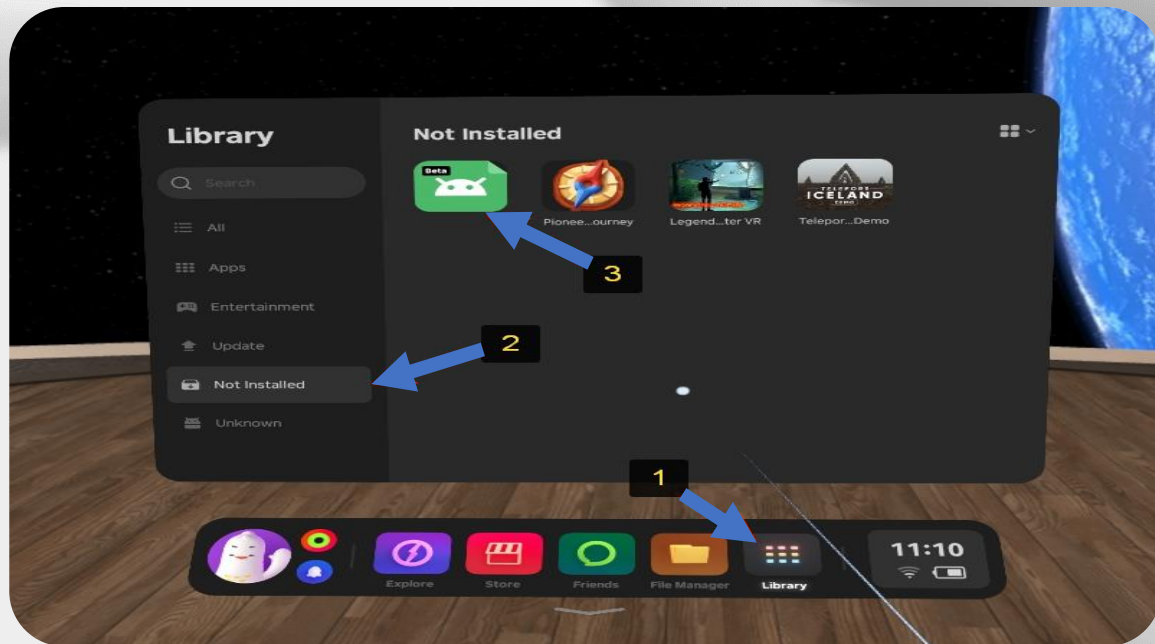



Log in the XR-Clinic Mobile

- The Profile key is provided in the welcome letter to your e-mail
- Enter the Profile key on the start screen of the XR-Clinic and click “Confirm”

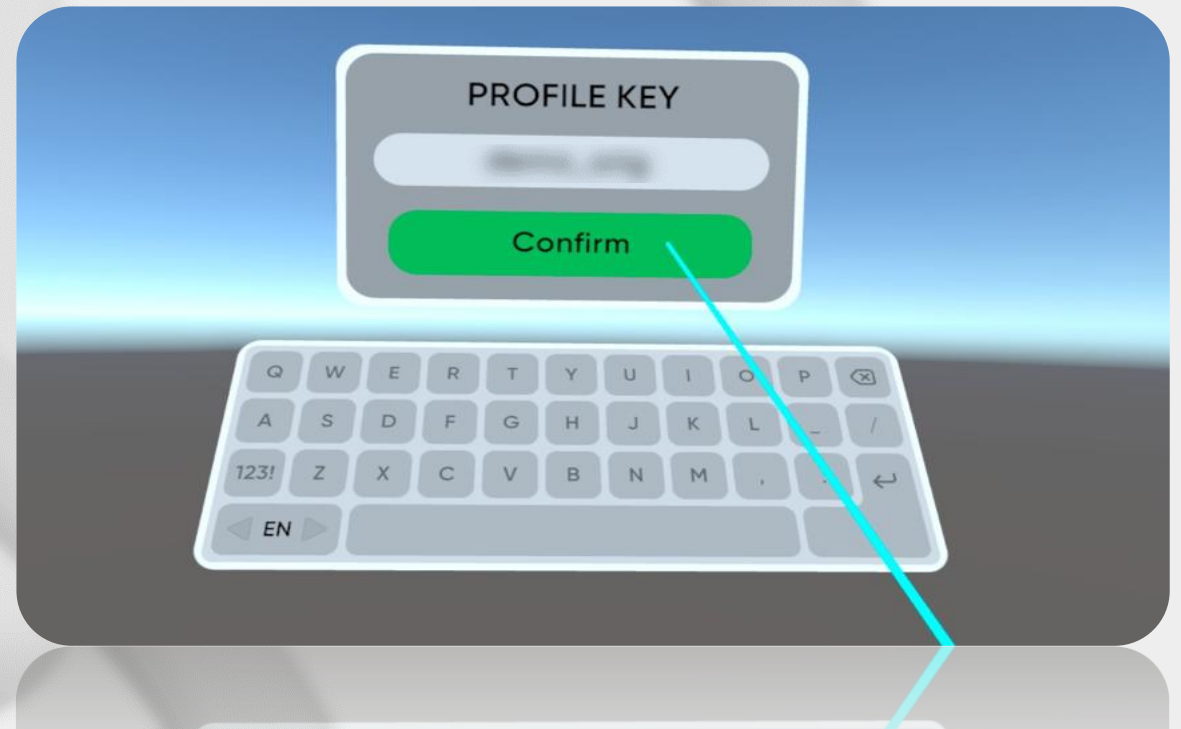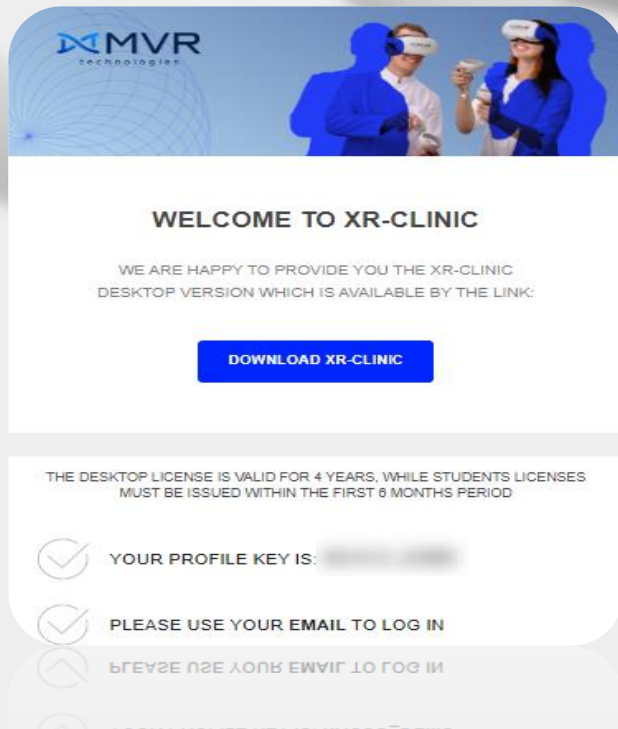


Log in the XR-Clinic Mobile

- Allow the application the access to file system (if needed)
- The license GUID should be confirmed in your Smart Center

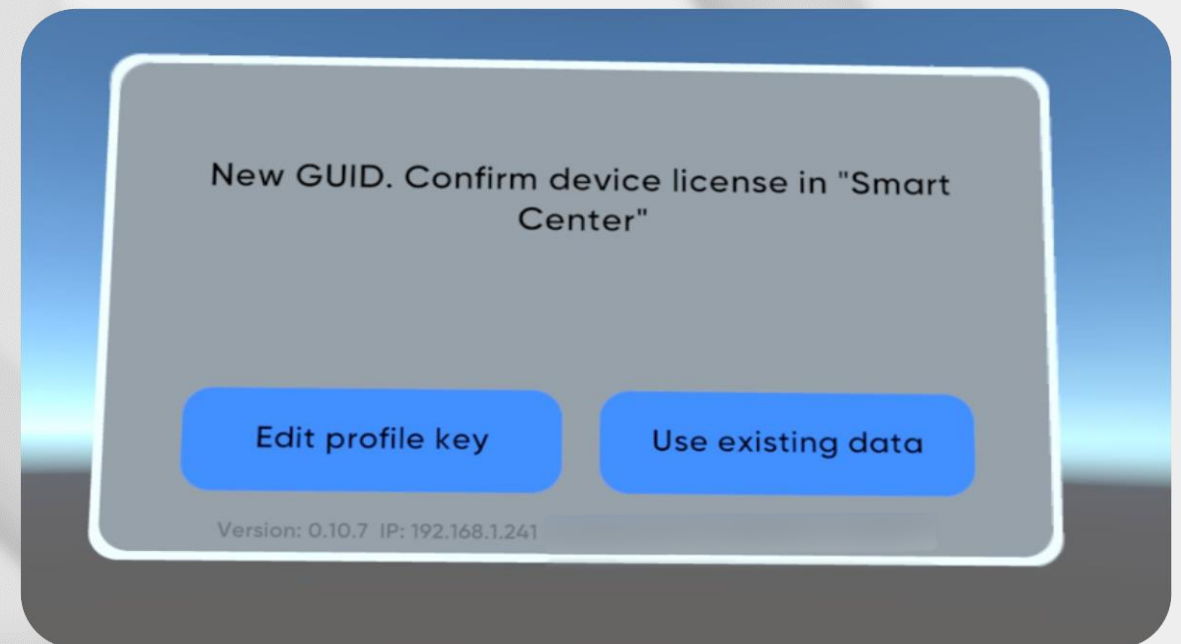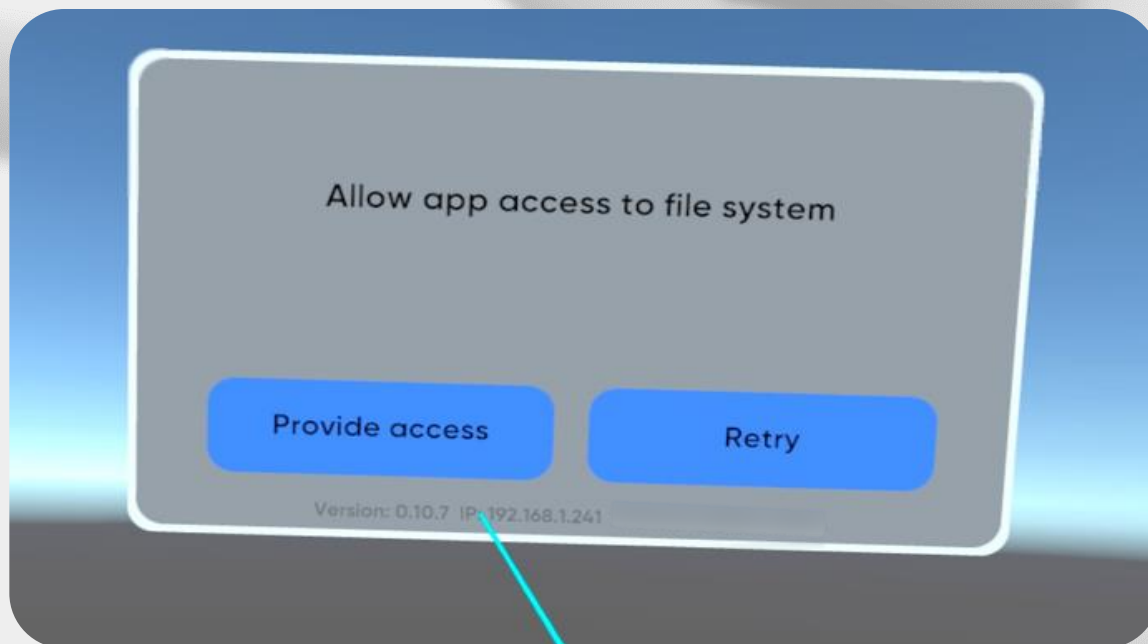


Log in Smart Center

- Go to: <https://case-manager.pro:8050>
- Enter login and password provided in the welcome letter and click “Sign in”
- Go to the “License” section

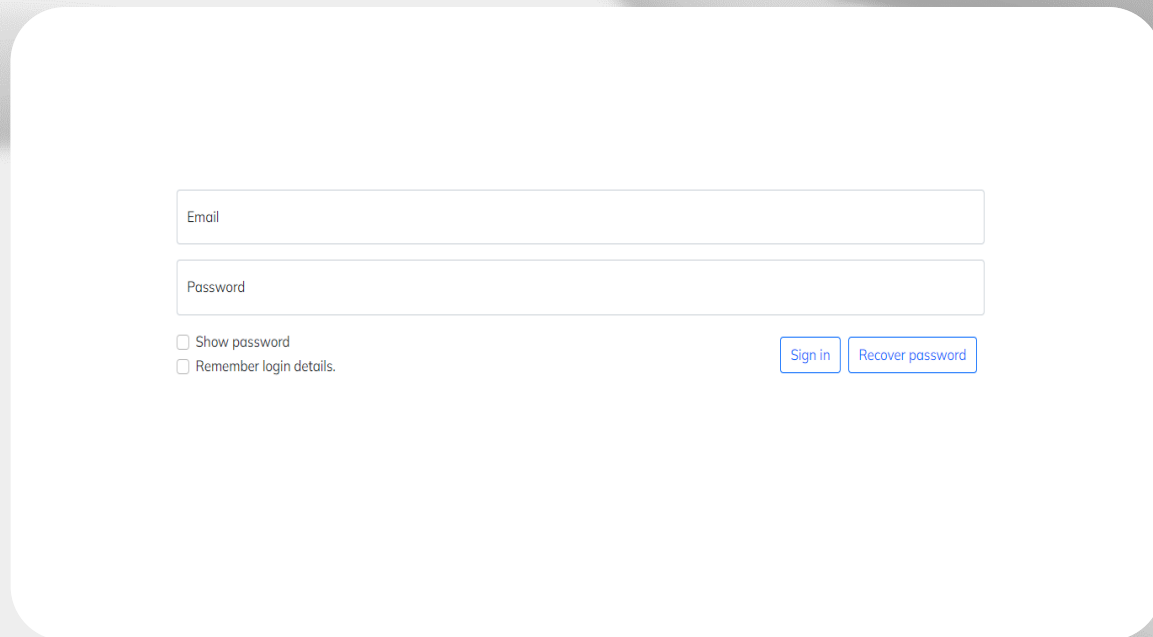

Form for logging into Smart Center. It includes fields for Email and Password, checkboxes for "Show password" and "Remember login details", and buttons for "Sign in" and "Recover password".

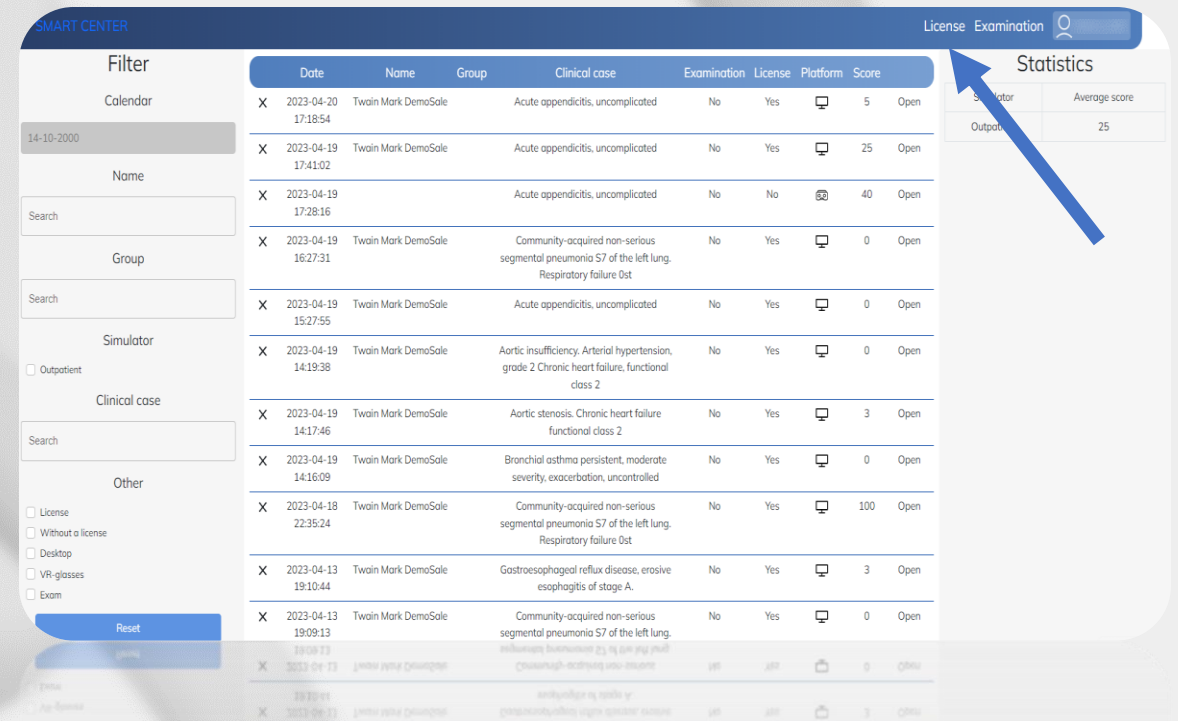

SMART CENTER

License Examination

Filter

Calendar

14-10-2000

Name

Search

Group

Search

Simulator

Outpatient

Clinical case

Search

Other

License

Without a license

Desktop

VR-glasses

Exam


Reset

Date	Name	Group	Clinical case	Examination	License	Platform	Score	
X 2023-04-20 17:18:54	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	5	Open
X 2023-04-19 17:41:02	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	25	Open
X 2023-04-19 17:28:16			Acute appendicitis, uncomplicated	No	No	📺	40	Open
X 2023-04-19 16:27:31	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung, Respiratory failure 0st	No	Yes	🖥️	0	Open
X 2023-04-19 15:27:55	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	0	Open
X 2023-04-19 14:19:38	Twain Mark DemoSale		Aortic insufficiency, Arterial hypertension, grade 2 Chronic heart failure, functional class 2	No	Yes	🖥️	0	Open
X 2023-04-19 14:17:46	Twain Mark DemoSale		Aortic stenosis, Chronic heart failure functional class 2	No	Yes	🖥️	3	Open
X 2023-04-19 14:16:09	Twain Mark DemoSale		Bronchial asthma persistent, moderate severity, exacerbation, uncontrolled	No	Yes	🖥️	0	Open
X 2023-04-18 22:35:24	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung, Respiratory failure 0st	No	Yes	🖥️	100	Open
X 2023-04-13 19:10:44	Twain Mark DemoSale		Gastroesophageal reflux disease, erosive esophagitis of stage A,	No	Yes	🖥️	3	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung,	No	Yes	🖥️	0	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung,	No	Yes	🖥️	0	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung,	No	Yes	🖥️	0	Open

Statistics

Simulator Average score

Output 25

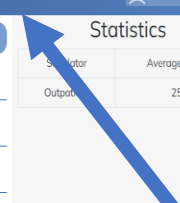

Confirm the Mobile license in Smart Center


- Your headset GUID is waiting for confirmation in the list of Mobile licenses
 - Click the “tick” icon to confirm the device
 - Headset GUID will appear in the list of Mobile licenses

Mobile license (VR device):

3 ID (VR device).

Confirmation required.

1	100-010	201-2a2	<input checked="" type="checkbox"/>	X
---	---------	---------	-------------------------------------	---




Mobile license (VR device):

3 ID (VR device).

1	100-010			
2	<u>201-2a2</u>			



Possible error

- If you skip the Smart Center confirmation step the notification will appear
 - Confirm the license in Smart Center and proceed with next steps

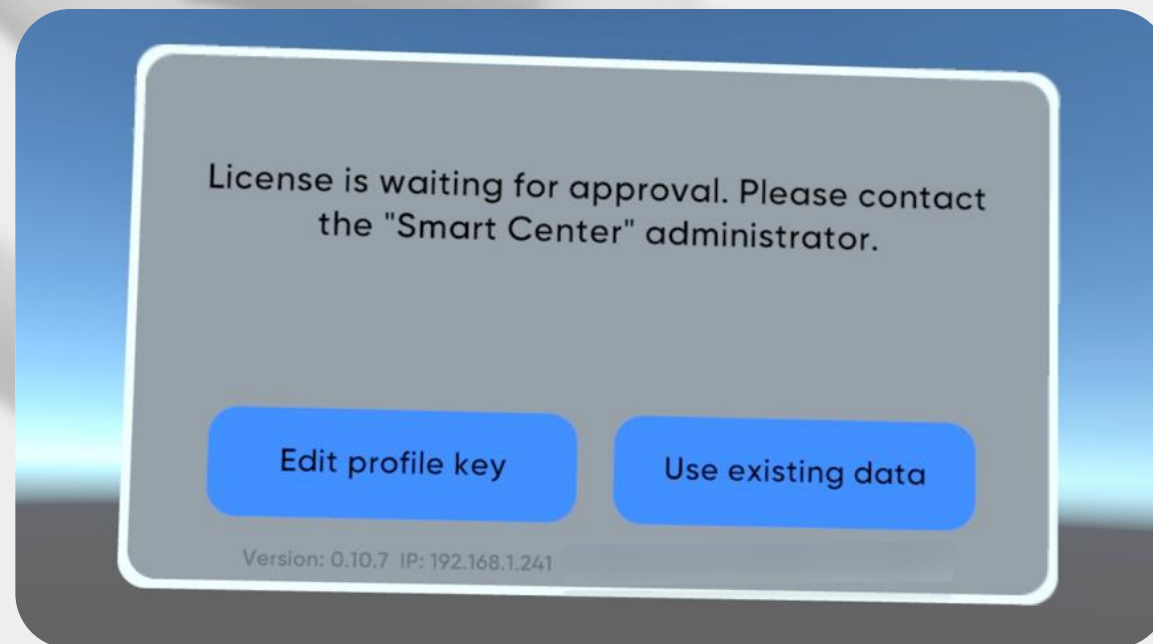



Finish XR-Clinic Mobile setup

- Click the “Edit profile key” button
- Enter your Profile key in the headset once again
- Download available cases from the Case Library

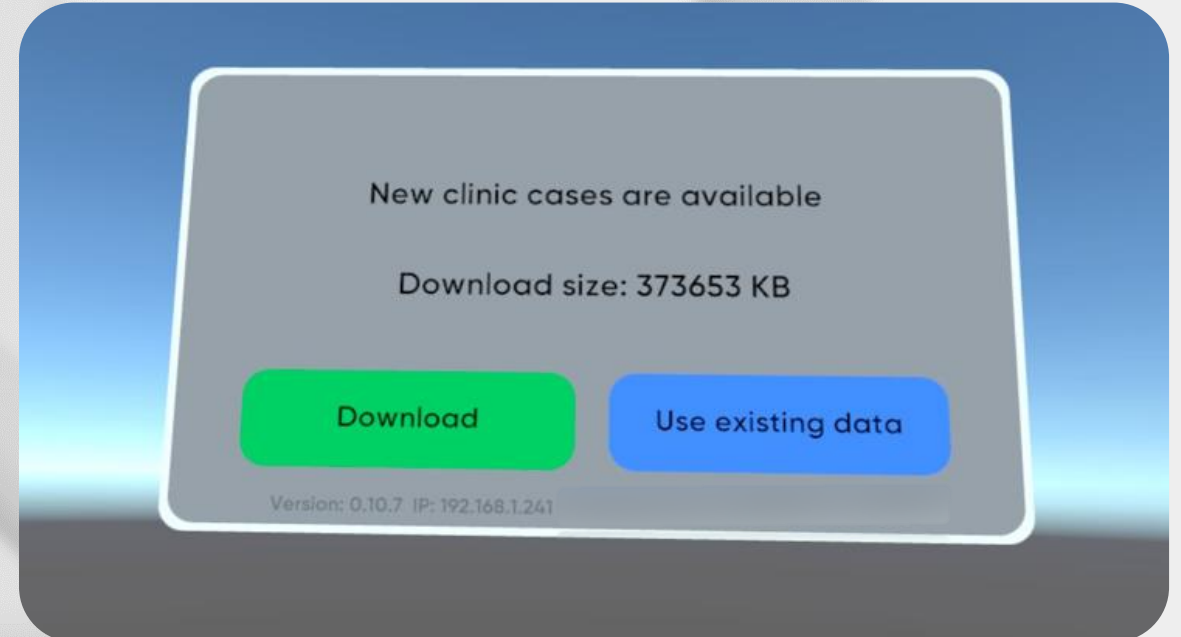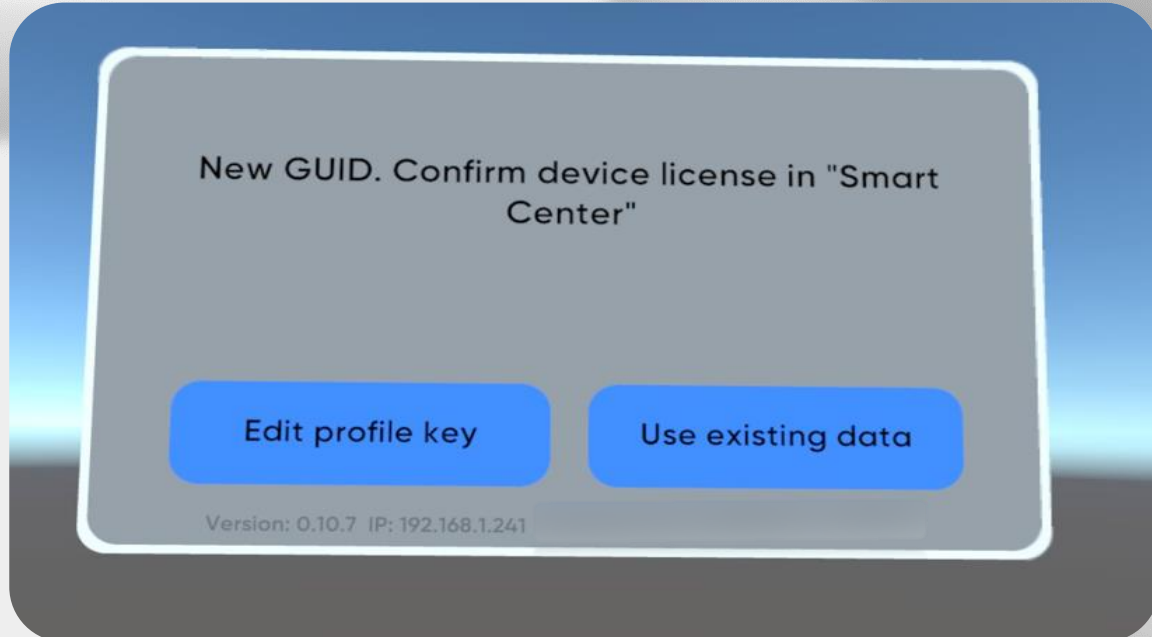

Finish the XR-Clinic Mobile setup

- Do not take your headset off while cases are downloading
- As soon as download is complete the XR-Clinic simulation will begin

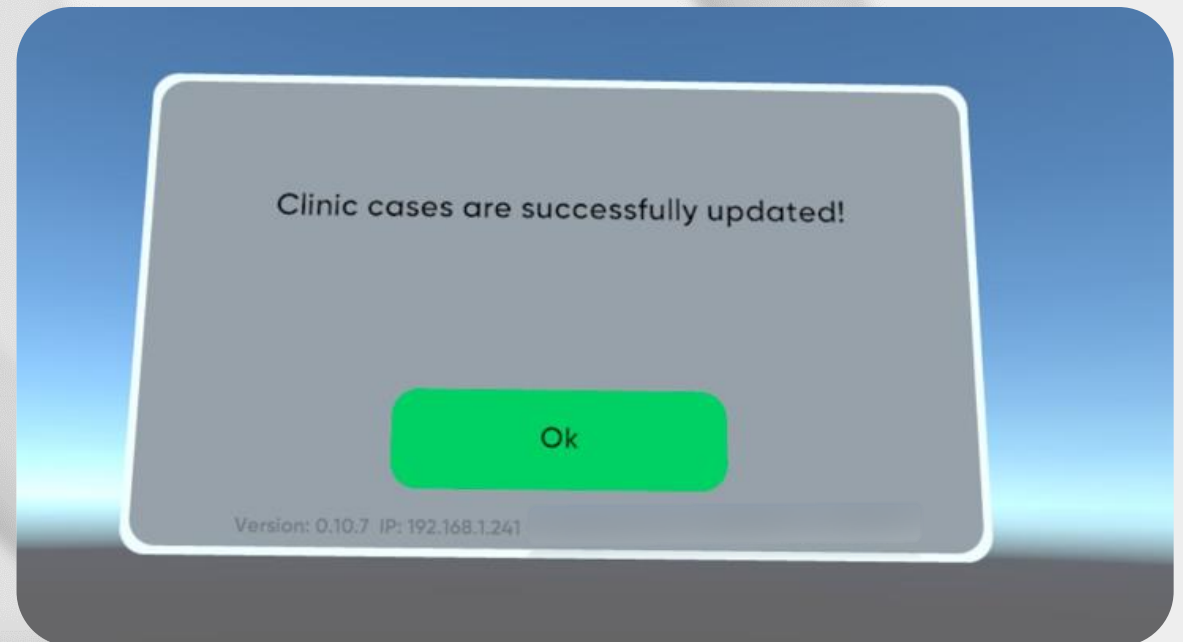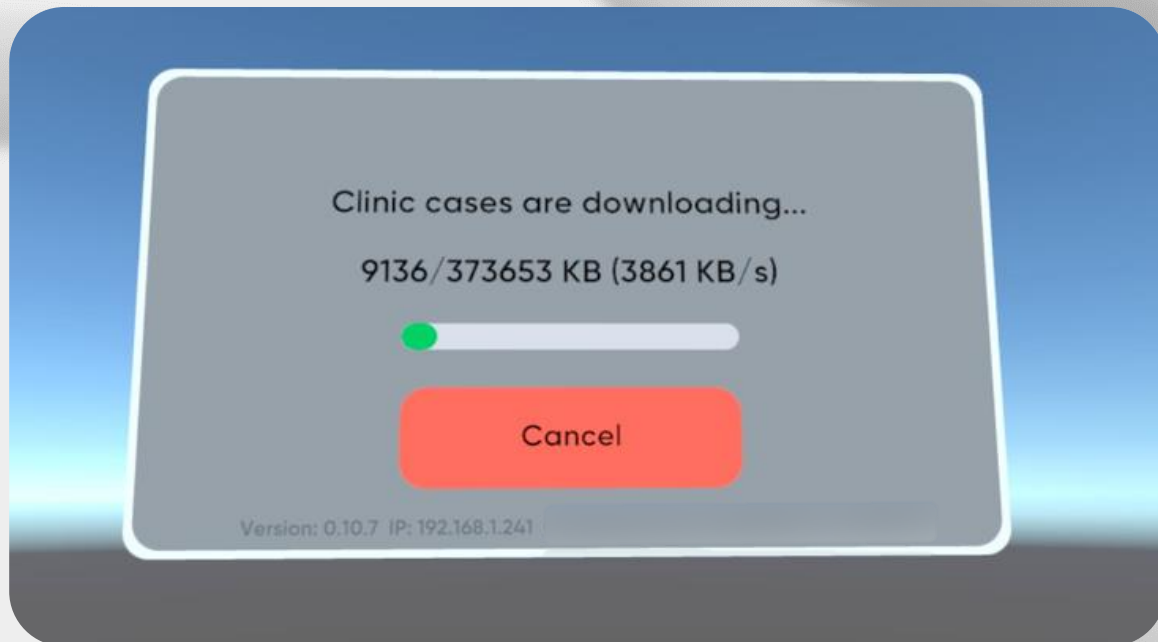

Be
READY

Technical support
service@mvredtech.com

Nasco
HEALTHCARE