

Babytherm® 8004/8010 Neonatal Open Care

Babytherm® is an intensive care Infant Warming System intended as a therapeutic device providing warmth for premature and full term neonates. Not only does it provide babies with a unique thermal environment, but it also allows you to enjoy outstanding ease of access so you can give them the best possible care.



Benefits

Optimal Thermoregulation

Babytherm works to deliver an optimal thermo-neutral environment for the neonate safe from cold or heat stress. The Babytherm's ceramic heater elements and golden reflectors provide even heat distribution over the entire bed area. So wherever you place the baby on the bed, your little patient will always feel comfortably warm. The Babytherm also provides a fully integrated heated conductive gel mattress to keep the baby warm at all times. Measuring a baby's central and peripheral (toe) temperature gives you an early warning of thermal instability or possible cold stress by observing the measured values on the integrated digital display.

Ease of Use

Our efforts to ensure you enjoy the best possible working conditions are reflected in a number of practical features. The control panel positioned directly beneath the radiant warmer is easy to reach so you can adjust the settings with no trouble at all. Another advantage is that you will easily be able to read all the essential information on the large, bright digital display from any position around the bed. The bright red central warning light at the end of the radiant warmer will alert you to an alarm condition even if you are working with another baby or in a different part of the room. A clear text message then provides you with all the details you require. The height of a Babytherm can be adjusted to suit your height, swivel cupboards are available to give you extra storage space for accessories and all the moving parts or control functions are easy to handle.

System Synergy

Together with the intelligent design of architectural systems, bed side monitoring, and gentle neonatal ventilation, you enjoy an integrated work environment that is ergonomically smart, intuitively simple, and perfectly accessible for improved communication, time savings, and decreased risk of errors across the NICU and beyond.

Intelligent Accessibility

The smartSWIVEL mechanism automatically focuses radiant warmth on the baby, even if the radiant heater is moved to one side to allow free working space for the caregiver. As a result, you don't need to worry about the baby cooling down or suffering cold stress during X-rays or other procedures because you can be sure of even radiant heat distribution at all times.

Developmental Care

Babytherm's developmental care concept comes in the smartSWIVEL which allows closeness for the family without compromising warming of the infant. Long dual temperature sensor cables facilitate the practice of Kangaroo Care. The working light features settings for day and night. Two brightness levels provide just the right lighting anytime. A central alarm light could be set up or a ramping alarm sound that is a low pitch melody set up away from the bed.

Accessories



D-50637-2012

Dräger VarioLux®

The Dräger VarioLux® offers cool, highly efficient, and variable intensity examination light specially designed for use in neonatal intensive care settings. Made for single-handed operation, it features adjustable color settings as well as a dimmer function.



D-7384-2009

Neonatal Care Accessories

Dräger accessories for neonatal ventilation and thermotherapy ventilate gently and effectively, reduce stress and help promote the development of the newborn with a wide range of accessories designed specifically for use with the smallest of patients.

Technical Data

BABYTHERM	8010	8004
STANDARD FEATURES		
SmartSWIVEL mechanism	•	•
Manual mode	•	•
Skin servo mode	•	•
Bright and soft light	•	•
Central Alarm	•	•
ThermoMonitoring™	•	•
Text messages	•	•
Bed tilt	•	•
Inner side walls (7 cm)	•	•
Heated gel mattress	•	
X-ray tray		•
OPTIONS		
Integrated phototherapy	•	•
Height adjustment	•	•
RS232 interface	•	•
Storage Drawers	•	•

Not all products, features, or services are for sale in all countries.

Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to www.draeger.com/trademarks to find the current status.

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
www.draeger.com

Manufacturer:

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23542 Lübeck, Germany

REGION DACH

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION MIDDLE EAST, AFRICA

Drägerwerk AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA PACIFIC

Draeger Singapore Pte. Ltd.
61 Science Park Road
The Galen #04-01
Singapore 117525
Tel: +65 6872 9288
Fax: +65 6259 0398
asia.pacific@draeger.com

REGION NORTH AMERICA

Draeger, Inc.
3135 Quarry Road
Telford, PA 18969-1042, USA
Tel +1 800 4DRAGER
(+1 800 437 2437)
Fax +1 215 723 5935
info.usa@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Indústria e Comércio Ltda.
Al. Pucurui - 51 - Tamboré
06460-100 - Barueri - São Paulo
Tel. +55 (11) 4689-4900
relacionamento@draeger.com
Panamá Comercial

Locate your Regional Sales Representative at:
www.draeger.com/contact

